NUK[®]
SEIT 1956 EST.

LUNA

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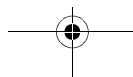
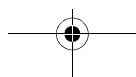
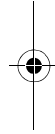
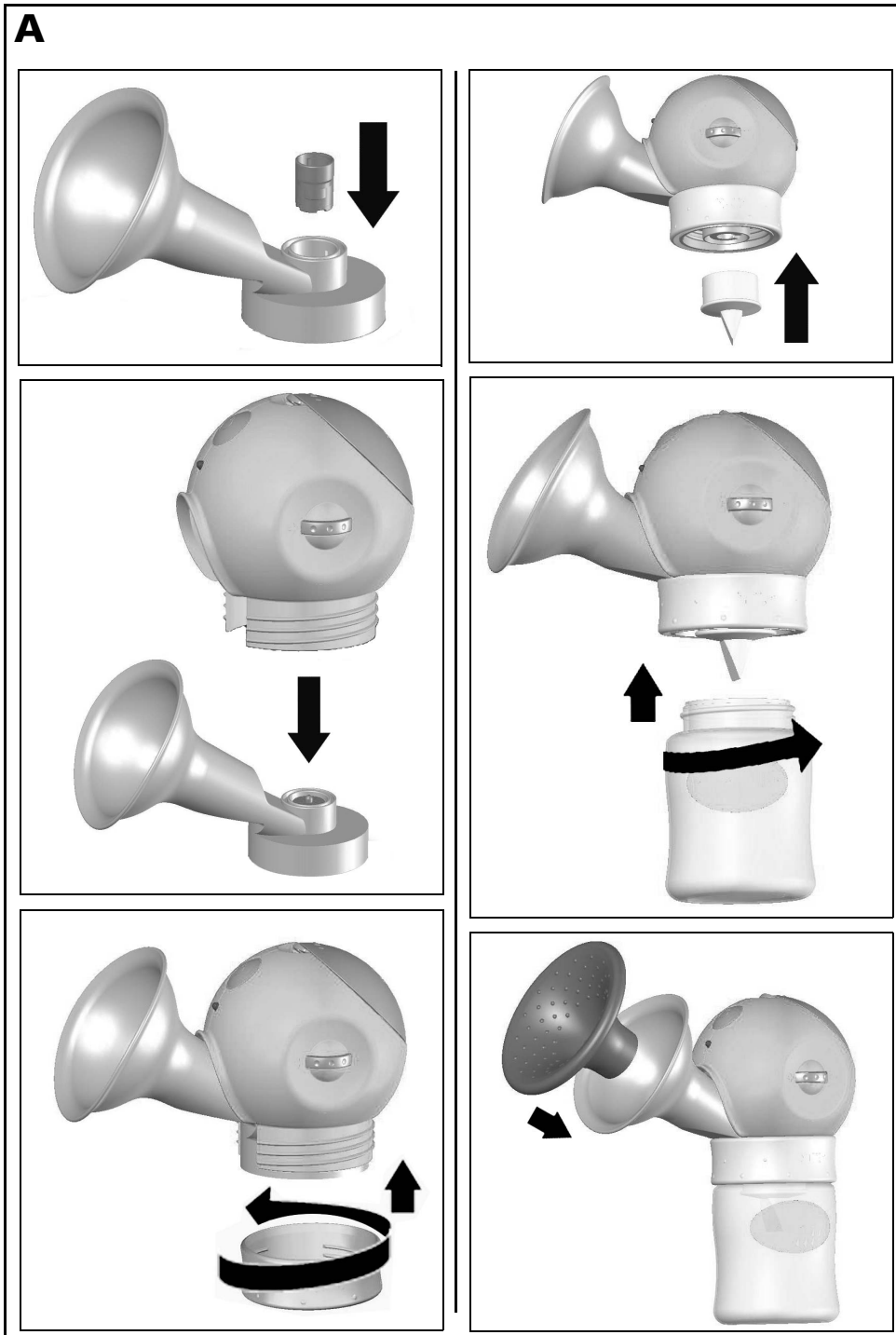
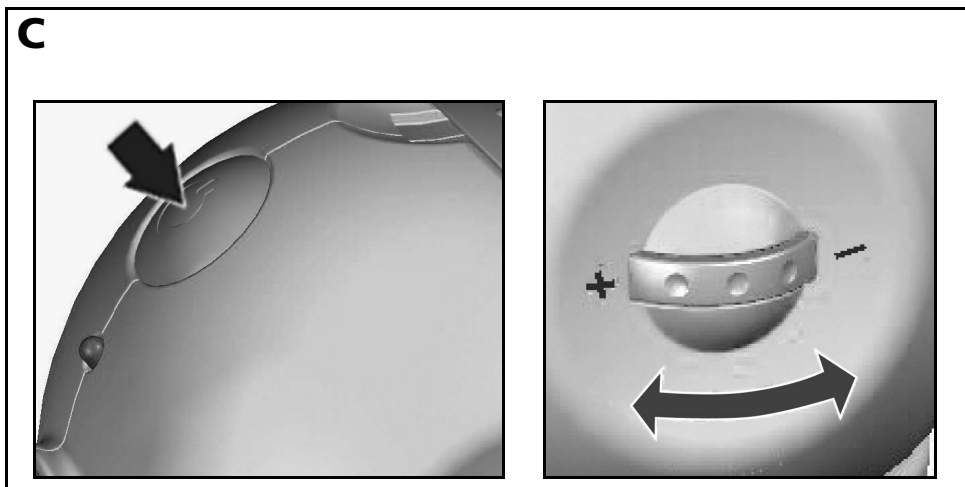
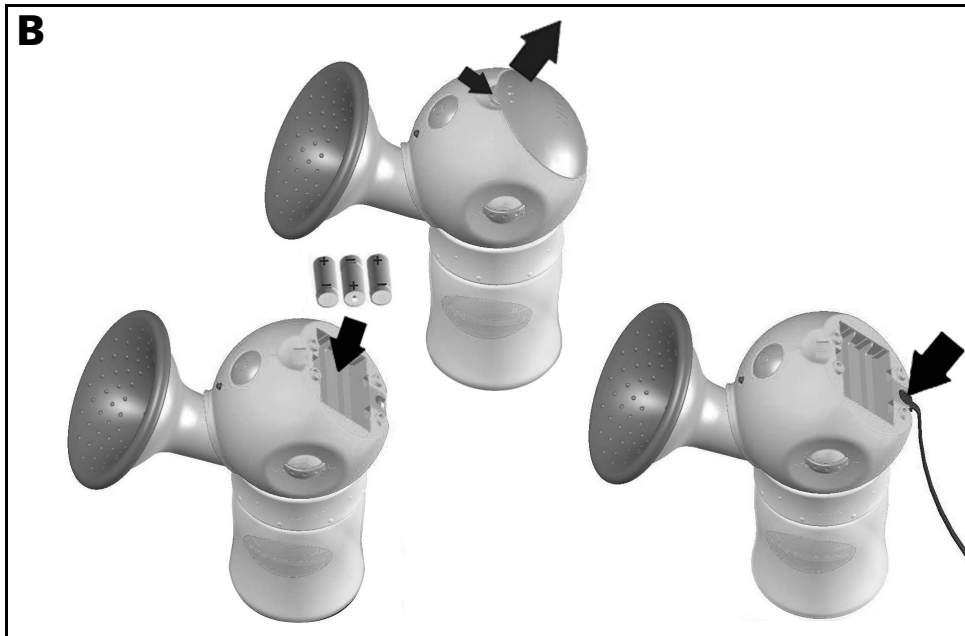
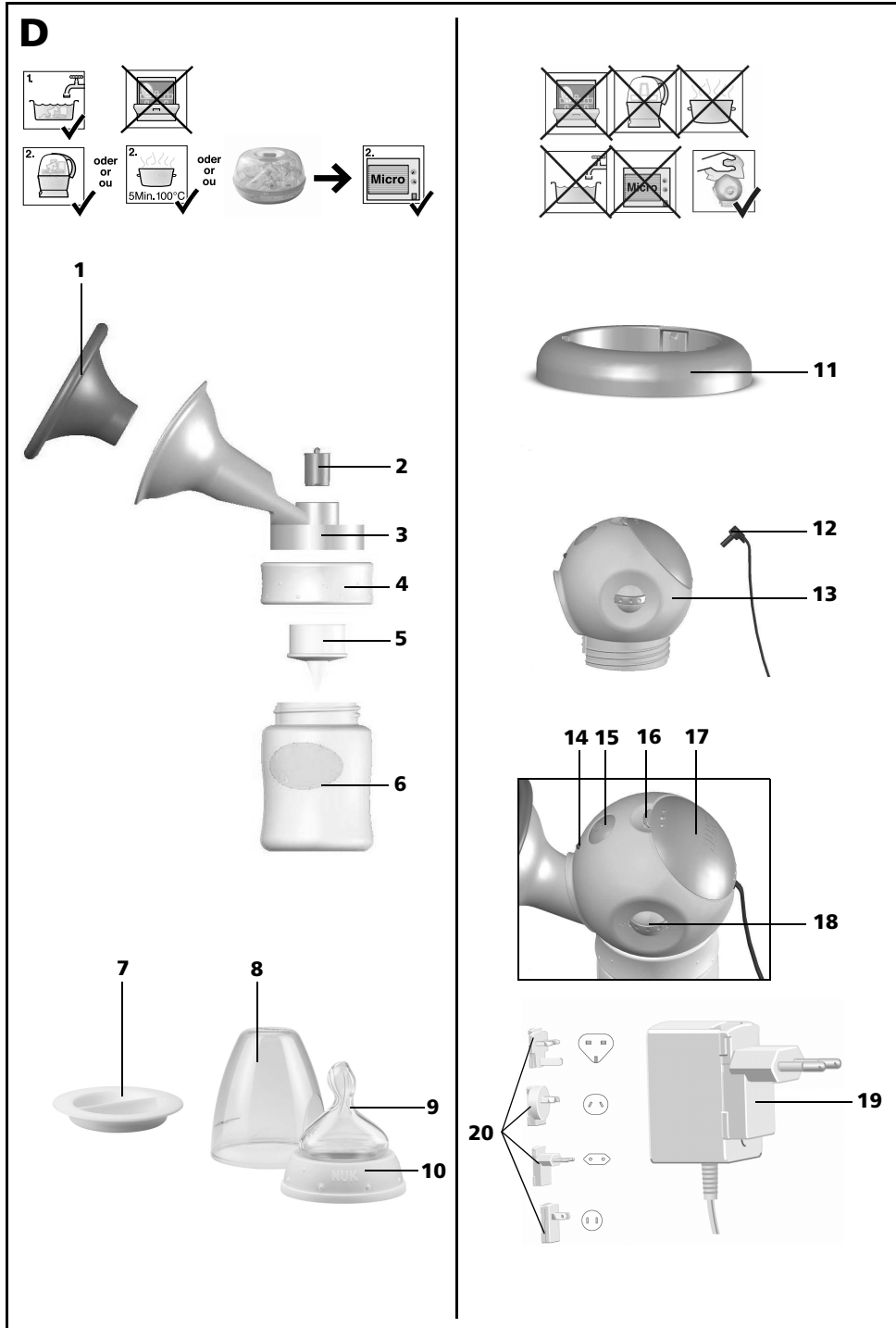
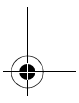
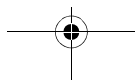
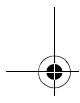
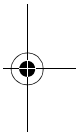
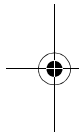
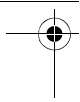
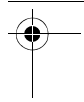
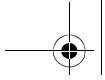
EMC-Guidance **127**






Contents

1. Important Information Before First Use	4
2. Description of Device	4
2.1 Intended use	4
2.2 Description of function	4
2.3 Contraindications	4
2.4 Side effects	4
2.5 Symbols on the pump	5
3. Safety Information	5
3.1 General information	5
3.2 Hygiene	6
3.3 NUK Breast Milk Container	6
3.4 NUK Teat	6
4. Designation of Components and functional Elements (Figure D)	7
5. Cleaning and Sterilising	8
5.1 After each use	8
5.2 Cleaning (Figure A in reverse order)	8
5.3 Sterilising	8
6. Assembly (Figure A)	9
7. Power supply (Figure B)	9
8. Operation (Figure C)	10
8.1 Expressing milk	10
8.2 Feeding expressed milk	11
8.3 Storing expressed milk	11
9. Tips for Successful Expressing	12
9.1 Frequency	12
9.2 Method	12
9.3 Quantity of milk	12
9.4 Let-down reflex	12
10. Hygiene Tips	13
11. Storing Milk	13
11.1 NUK Breast Milk Container	13
11.2 Important notes for handling breast milk	14
12. Troubleshooting	15
13. Technical Data	16
14. Disposal	16
15. Warranty	17
16. Warranty Certificate	17

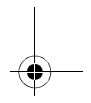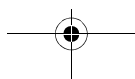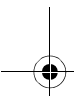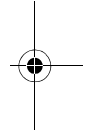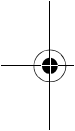Dear Mother,

Breast milk is the best food for a baby during the first six months of its life. To enable you and your baby to enjoy every moment of breastfeeding, NUK has developed a range of practical and comfortable breastfeeding aids for the systematic support of breastfeeding.

The NUK Luna electric breast pump is intended as an aid for you and your baby if

you are unable to breastfeed naturally on a temporary or permanent basis. Please take a few minutes to read carefully through these operating instructions before using your NUK Luna Electric Breast Pump for the first time.

We wish you and your baby all the best.

1. Important Information Before First Use

Attention! Clean and sterilise parts **1** to **10** before using the NUK Luna Electric Breast Pump for the first time (see "5. Cleaning and Sterilising" on page 8).

2. Description of Device

2.1 Intended use

The NUK Luna Electric Breast Pump is an aid to expressing and collecting breast milk:

- in the workplace or when baby is away, at home or when travelling
- to maintain milk production during a break in breastfeeding
- to relieve engorgement
- to increase milk production

Once expressed, breast milk can be collected, stored, frozen, warmed and fed to your baby in the NUK Breast Milk Container.

The NUK Luna electric breast pump is intended for personal use. It is not suitable for commercial use.

2.2 Description of function

The NUK Luna Electric Breast Pump is an electric suction pump and offers the following benefits:

- a 2-phase pumping rhythm to simulate a baby's natural sucking rhythm when breastfeeding

- fully adjustable suction strength
- ergonomically-shaped breast shield
- soft silicone cushion
- both mains and battery operation (mains adapter supplied).

2.3 Contraindications

In the event of certain breast problems (infections, open cuts), the NUK Luna Electric Breast Pump should only be used after consulting the doctor or health visitor supervising treatment.

2.4 Side effects

If at any time you experience an unexpected reaction, please consult your midwife, health visitor, doctor or pharmacist.

2.5 Symbols on the pump

In the motor unit (battery compartment)	
Symbol	Meaning
	• When inserting batteries, check for correct pole alignment
	• Do not dispose of batteries or the pump with household waste
	• Refer to the operating instructions
	• Refer to accompanying documents
	• Serial number

On the motor unit	
BP5	• Model name
	• Complies with all the requirements of applicable EC directives
	• Electrically protected device, Protection Class II
	• Electromedical device, Type BF
	• Refer to accompanying documents
	• Direct current

3. Safety Information

3.1 General information

Breastfeeding is best for your baby.

For your own safety please observe the following:

- Please carefully read the operating instructions before using the pump. These instructions are part of the device and should be available at all times.
- Use the NUK Luna Electric Breast Pump and accessories only for the intended use described (see "2.1 Intended use" on page 4).
- Electrical medical devices such as this one may only be used by adults.
- The breast pump may only be mains-operated with the mains adapter supplied.
- Check that the mains voltage required for mains adapter **19** (see rating plate) is compatible with your mains voltage.
- Never immerse motor unit **13** in water, as there is a risk of electrocution and the motor unit might be damaged by water penetration.
- Do not expose the device to direct sunlight, as UV rays and overheating may make plastics brittle.
- To avoid risk of choking, keep small parts out of the reach of children.
- We recommend removing ointments/creams containing grease or oil from the breast before expressing, as these may damage the surface of the breast shield.
- Only use original NUK accessories, as only these guarantee that the pump will function correctly.
- The batteries must be removed when flat or if the device is not going to be used for an extended period, as leaking batteries may damage the device.
- It is recommended not to use rechargeable batteries, as these usually have a reduced charge

capacity, with the result that the pump will not be fully functional after a very short time.

- Please note that any claim under warranty or liability will be void if accessories are used which are not recommended in these operating instructions, or if any parts other than original spare parts are used for repairs. This also applies to repairs carried out by unqualified persons.
- Using articles from a different manufacturer may cause malfunctions.
- If the mains connection lead for this device is damaged, it must not be repaired. Instead, the entire mains adapter must be replaced by an original NUK mains adaptor (see "13. Technical Data" on page 16) in order to avoid potential danger.
- In the event of any malfunction, the pump may only be repaired by the manufacturer, otherwise your warranty will be invalidated.
- Never open motor unit housing **13** – this will invalidate the warranty.
- If the breast milk is for a premature baby, it is essential to follow your doctor's instructions.

3.2 Hygiene

- When handling breast milk, hygiene is of particular importance to ensure your baby's healthy development.
- Remove all body jewellery (e.g. piercings) before using the breast pump.
- Please refer to the section entitled "5. Cleaning and Sterilising" on page 8 to prevent infection or bacterial contamination (build-up of bacteria)

and to prevent the device from becoming damaged.

- **Only a breast pump which has been thoroughly cleaned will achieve full suction strength.**

3.3 NUK Breast Milk Container

- If the breast milk container is damaged on the inside (e.g. by scratches), it should be replaced for hygiene reasons.
- Any individual parts not currently in use must be stored out of the reach of children (choking hazard).
- Tea or juices may discolour breast milk container and accessories.
- Only administer food and drink in the quantity and consistency recommended by the respective manufacturer.
- Do not microwave closed breast milk containers filled with feed (risk of explosion/material damage). Heating in this way could cause the contents of the breast milk container to explode. Uneven heating carries a risk of scalding.
- NUK Breast Milk Containers and teats are only compatible with the associated NUK FIRST CHOICE system.

3.4 NUK Teat

- Clean and sterilise the teat thoroughly before using for the first time and before each subsequent use.
- The NUK Steam Steriliser is recommended for sterilising.
- When boiling, check that there is sufficient liquid in the pan to prevent damage (5 minutes boiling time is sufficient).


- Washing in a dishwasher is not recommended, as detergent may damage or deform the teat.
- For your child's safety and health:
WARNING!**
- Store in a hygienic environment and away from direct light sources.
 - Always check food temperature before feeding.
 - Always use this product with adult supervision.
 - Never use feeding teats as a soother. Continuous and prolonged sucking of fluids will cause tooth decay.
 - Do not dip the teat in sweetened liquids or medicine.
 - Check the teat before each use by pulling it in all directions - especially when children are teething. Discard the teat at the first sign of tearing or biting damage, as there is a risk of small parts becoming detached and causing choking.
 - For reasons of hygiene and safety, teats should be replaced every 1 to 2 months.
 - Use teat only in combination with the NUK Breast Milk Container or NUK FIRST CHOICE Bottles. Both the NUK Teat and the NUK Breast Milk Container comply with EN 14350.

4. Designation of Components and functional Elements (Figure D)

- | | |
|--|--|
| 1 Silicone cushion | 12 Power supply plug |
| 2 Valve | 13 Motor unit |
| 3 Breast shield | 14 Warning lamp when voltage too low
(batteries need changing) |
| 4 Connection ring | 15 On/Off button
–for switching on and off
–for switching from Phase 1 to Phase 2 |
| 5 Lip valve (+ 1 replacement lip valve) | 16 Battery compartment catch |
| 6 Breast milk container | 17 Battery compartment lid |
| 7 Sealing disc | 18 Suction strength controller |
| 8 Protective cap | 19 Mains adapter |
| 9 Teat | 20 Interchangeable plugs |
| 10 Screw ring | |
| 11 Stand for NUK breast milk container | |

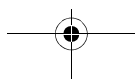5. Cleaning and Sterilising

Hygiene is the top priority when handling breast milk and all parts which come into contact with it. You should therefore only use a pump that is clean and sterilised.

8. Wipe motor unit **13** with a damp cloth.

Caution! Never immerse the motor unit and the mains adapter in water, as this will damage the electronics.

5.1 After each use

- Parts **1** to **10** (not parts **11** to **20**) should be washed thoroughly in water and sterilised **immediately after every use**.

5.2 Cleaning (Figure A in reverse order)

Caution! Unplug mains adapter **19** from the wall socket before cleaning.

Caution! Clean lip valve **5** carefully. We recommend just rinsing off the lip valve and boiling it to sterilise. Cleaning implements may lead to damage.

1. If you are using silicone cushion **1**, release this fully from breast shield **3**.
2. Unscrew motor unit **13** and connection ring **4** from breast milk container **6**.
3. Remove lip valve **5** from the underside of the breast shield **3**.
4. Unscrew connection ring **4** from motor unit **13**.
5. Take motor unit **13** off breast shield **3**.
6. Remove valve **2** from breast shield **3**.
7. Wash parts **1** to **10** (not parts **11** to **20**) under running water and rinse.

You can also use a little mild detergent (e.g. NUK Baby Bottle Cleanser).

Milk residues may damage the material and result in microbial growth.

5.3 Sterilising

Before sterilising, clean as described in the section entitled "5.2 Cleaning (Figure A in reverse order)".

Important! Under no circumstances sterilise the stand, the motor unit or the mains adapter (parts **11** to **20**), as this would deform the material and/or damage the electronics.

- Use a NUK Steam Steriliser. Follow the operating instructions associated with the device. Alternatively,
- boil the parts for 5 minutes in a pan of vigorously boiling water. Ensure there is sufficient water in the pan to avoid damage.
- Only the NUK Microwave Steam Steriliser should be used for sterilising in a microwave. Follow the manufacturer's instructions when using any sterilisers or sterilising solution in order to prevent damage to the product.

Important! The sterilised parts must be thoroughly cooled down and completely dry before being reassembled and used again.

6. Assembly (Figure A)

1. Insert valve **2** in breast shield **3** with the knob pointing upwards.
2. Put motor unit **13** on breast shield **3**.
3. Screw connection ring **4** onto motor unit **13** from underneath.
4. Insert lip valve **5** into assembled breast shield **3** from underneath.
5. Screw assembled motor unit **13** and breast milk container **6** together.
6. If desired, put the silicone cushion on the breast shield and press on firmly.

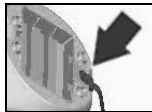
Note: Make sure that the silicone cushion fits closely to the breast shield as the pump can otherwise draw in air, which will prevent it from working properly.

7. Power supply (Figure B)

7.1 Mains operation

Only use the mains adapter supplied for **mains operation**.

1. Open the battery compartment at catch **16**.
2. Insert plug **12** of mains adapter **19** in the socket in the battery compartment.

3. Close the battery compartment:

push the two "hooks" of the lid into the relevant openings in the motor unit and then close the lid so that catch **16** engages audibly.

Note: The device may not be operated without the battery compartment lid.

4. Plug the interchangeable plug **20** suitable for your power supply onto the mains adapter. You can find out which plug is suitable for your country from the section which follows.
5. Plug mains adapter **19** into a wall socket.



7.2 Interchangeable plugs

The mains adapter is supplied with an EU plug. Additional interchangeable plugs are also included. You can use these to operate your breast pump when abroad as follows:


		Commonwealth, Africa, Asia, Great Britain, Middle East etc.
		Australia, New Zealand, China
		Europe, South America, Africa etc.
		North, Central and South America, Japan, South East Asia etc.

Changing the plug

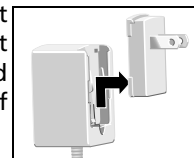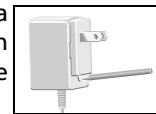1. Using an item such as a ball-point pen, press on the release catch on the plug.
2. Slide the plug about 1.5 cm upwards at the same time, and then take it off towards you.

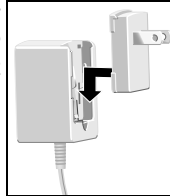3. Position the interchangeable plug so that its grooves meet the contacts on the mains adapter.

4. Slide the plug downwards until the locking mechanism audibly engages.

7.3 Battery operation

For **battery operation** you need 3 batteries of the AAA 1.5 V type. We do not recommend operating the device with **rechargeable batteries**. Rechargeable batteries usually have less charge capacity than disposable batteries, with the result that the pump fails to achieve full output after a very short time. Moreover, any rechargeable batteries inserted will not be recharged. When you are operating the pump using batteries, the warning lamp **14** will warn you that power is too low. You need to change the batteries as soon as this lamp is red. If the batteries become

completely discharged, warning lamp **14** is no longer able to come on, as the pump switches off because there is not enough power.

1. Open the battery compartment at catch **16**.
2. Insert the batteries, ensuring that they are correctly positioned in the battery compartment according to the + and - symbols!
3. Close the battery compartment:

push the two "hooks" of the lid into the relevant openings in the motor unit and then close the lid so that catch **16** engages audibly.

Note: The device may not be operated without the battery compartment lid

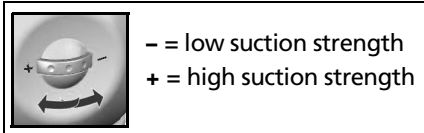
Caution! The batteries should be removed if the device is not going to be used for an extended period to prevent leaking batteries causing damage to the device.

8. Operation (Figure c)

8.1 Expressing milk

Caution! As hygiene is of particular importance, please always wash your hands before expressing milk and observe the advice in section "10. Hygiene Tips" on page 13.

Important!


1. To start with, turn the suction strength controller **18** to the lowest suction

strength by rotating the ring towards the "-" symbol. You can then easily increase suction strength as you express.

2. Switch on the breast pump by pressing On/Off button **15** once.
3. Position breast shield **3** on your breast so that your nipple is in the centre of the shield. Gently press the breast shield to ensure a good seal when expressing. **Hold the pump as upright as possible as you do this.**

Note: If you hold the pump at too great an angle, or if the pump topples over, it will switch off automatically.

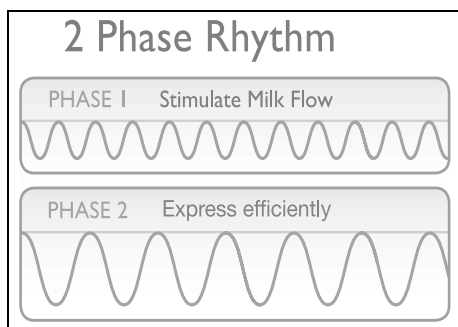The pump has a 2-phase pumping rhythm to simulate a baby's natural sucking rhythm when breastfeeding. The phases can be selected individually.

Phase 1: The sucking rhythm is rapid and stimulating during this phase to stimulate the milk flow.

Phase 2: To switch from Phase 1 to Phase 2, press On/Off button **15** a second time. During Phase 2, the pump sucks in a slow and intensive rhythm ensuring maximum milk flow. The combination of the 2 phases ensures efficient expressing.

4. Adjust the suction strength using the suction strength regulator **18** until the optimum suction strength is attained.

If the suction strength is too weak, turn the suction strength regulator **18** towards the "+" symbol.

If the suction is too strong, turn it towards the "-" symbol.

Note: Please note that the breast pump has a higher suction strength in Phase 2, so do not set suction strength too high at first.

Caution! To maintain function, express only enough milk so that when collected in the breast milk container, it does not contact the bottom edge of lip valve **5**.

5. Press On/Off button **15** a third time to complete the expressing operation and switch off the pump.

Note: If you want to interrupt expressing without switching off the pump, simply place a finger between your breast and the breast shield to break the vacuum.

Note: You can switch to the desired phase at any point by pressing On/Off button **15** again.

6. After expressing, place the milk pump in stand **11** otherwise the pump might topple over.

7. If you have been operating the device using mains adapter **19**, take mains adapter **19** out of the wall socket to disconnect the device completely from the power supply.

8. Unscrew breast milk container **6** from connection ring **4** /the motor unit. Ensure you keep the breast milk container **upright** in the process.

9. Immediately after expressing, wash the individual parts (see "5. Cleaning and Sterilising" on page 8).

8.2 Feeding expressed milk

Screw teat **9** and screw ring **10** onto breast milk container **6**.

8.3 Storing expressed milk

1. To store the milk, secure the breast milk container **6** with the sealing disc **7** provided, which should be screwed

onto the breast milk container using the screw ring **10**.

Caution! It is essential to follow the instructions in the section entitled "11. Storing Milk" on page 13.

2. Use a freezer pen to write the date and time of expressing on a label.
3. Stick the label onto the NUK breast milk container.

9. Tips for Successful Expressing

Correct expressing takes practice. You will first need to get used to your new pump.

9.1 Frequency

How often you need to express depends on the reason for expressing. If you will be apart from your child and/or want to sustain milk production, the natural breastfeeding rhythm should be maintained. You should therefore express 6 to 8 times daily, including once during the night. In all other cases you should express entirely according to your personal wishes and requirements.

9.2 Method

Milk should be expressed from each breast in turn. The 2-phase pumping system of this electric breast pump simulates a baby's natural sucking rhythm.

In Phase 1, the suction rhythm is rapid and stimulating to trigger the let-down reflex. This simulates a baby's sucking which consists of rapid sucking movements at the start.

During Phase 2, the breast pump sucks in a slow and more intensive rhythm, ensuring maximum milk flow. This corresponds to your baby's drinking behaviour. As soon as a baby's initial thirst is quenched, it switches to a calm, slow and intensive sucking action to drink its fill.

If the milk flow drops:

- Change breast

- or try to stimulate the let-down reflex again by selecting Phase 1 again. To do this, press On/Off button **15** twice.

The amount of time needed to express milk will vary considerably from one person to another.

9.3 Quantity of milk

Don't be surprised if it takes a little while for the milk to start to flow properly at first. This is not unusual. In fact it is quite normal because

milk production adapts itself to daily requirements.

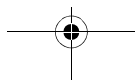
This means that the quantity of milk produced may vary considerably.

- Ensure you drink plenty of liquid yourself.
- Do not set suction strength too high. A higher suction strength does not increase the quantity of milk produced.

9.4 Let-down reflex

During natural breastfeeding, placing the baby on the breast stimulates the let-down reflex and milk begins to flow. You can also stimulate the let-down reflex as follows:

- Find yourself a quiet place to sit, where you can relax fully when using the breast pump.
- Adopt a comfortable and relaxed sitting position.

- Place a warm compress (e.g. NUK Cool & Warm Gel Breast Compress) on your breast for a few minutes, to relax the tissue, and massage your breast.
- Ensure you have eye contact with your baby. If this is not possible, look at a photo of your baby.
- The let-down reflex triggers milk flow in both breasts simultaneously. You can use the NUK Breast Milk Collection Shell to collect breast milk leaking from the breast you are not currently expressing.

10. Hygiene Tips

Hygiene is especially important when using the breast pump. This applies to all the individual pump parts as well as to your own personal hygiene. We recommend

washing your nipples in clean water daily. Soap should preferably not be used, as it may irritate the nipples.

11. Storing Milk

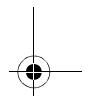
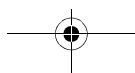
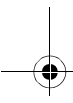
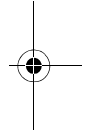
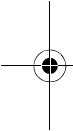
11.1 NUK Breast Milk Container

If the breast milk is not used immediately, you must seal the filled breast milk container firmly and store it in the refrigerator or freeze it.

Caution! Do not place the entire NUK Luna electric breast pump in the refrigerator, just the breast milk container **6** with screw ring **10** and sealing disc **7**.

Breast milk may be chilled, frozen and defrosted in the breast milk container. The container may also be used for warming and feeding. It is important that you follow the instructions about handling breast milk which is provided below.

Note: When freezing breast milk, do not fill the container with more than 150 ml (5 fl oz), as breast milk expands as it freezes and the container could burst.

11.2 Important notes for handling breast milk

Breast milk	Storage location	Storage periods
Freshly expressed	At room temperature	Max. 6-8 hours
	Refrigerator (not in door)	72 hours at 4 °C
	Refrigerator freezer compartment	2 weeks
	Upright/chest freezer	6 months at -18 °C
Defrosted, unopened	Refrigerator (not in door)	24 hours at 4 °C
Defrosted, opened	Refrigerator (not in door)	12 hours at 4 °C
Defrosting milk	Gradually in refrigerator	Approx. 24 hours at 4 °C
	At room temperature followed by immediate feeding	
	If time is short, defrost container with breast milk in lukewarm water (max. 37°C), then feed immediately.	

- **Caution!** If transporting chilled breast milk make sure that it stays chilled at all times, otherwise the milk could go off and be harmful to your baby. Use a cooler bag with ice packs, for example.
 - **Caution!** Do not use a microwave to defrost breast milk, as this can destroy valuable nutrients and damage the container material (risk of explosion). Uneven heating may result in scalding!
 - To achieve the correct drinking temperature of approx. 37 °C, put the container of defrosted breast milk in the baby food warmer.
- Note:** The container must be open to prevent pressure build-up as the milk warms up.
- Shake the milk gently before feeding so that milk and milk fat are properly mixed again.
 - Defrosted, warmed or unused milk must be disposed of and not refrozen.
 - If you express breast milk more than once within a period of 24 hours, the freshly expressed milk may be added to the previously cooled breast milk. However freshly expressed milk must be cooled down before being added to milk that is already cool. **The temperatures must be the same (cold on cold).**
 - Consult your health visitor, midwife, doctor or pharmacist for detailed information about long-term storage.
 - You can find more information about breastfeeding on our NUK website (www.nuk.com).

12. Troubleshooting

Problem	Reason	Remedy
Pump does not work	No power supply	For battery operation: check whether batteries are correctly inserted. For mains operation: check that the mains plug and mains adapter are correctly inserted.
	Lip valve missing, incorrectly positioned or damaged	Check that the lip valve is correctly positioned. If it is damaged, use the replacement lip valve supplied.
	Pump is being held at an angle	Hold the pump upright. If the pump is held at an angle, it switches off automatically
	Breast milk container is full and lip valve is already contacting the expressed milk	Replace breast milk container
	Silicone cushion incorrectly positioned and not properly flush with the breast shield	Ensure that the silicone cushion is completely flush with the breast shield
	Screw ring not tightened up enough	Tighten screw ring up more
	No vacuum can be built up	Ensure that the breast shield is completely flush with the breast
	Pump suction is weak	Low battery power
Suction strength is set too low		Turn the suction strength controller in the "+" direction to achieve optimum suction strength
Silicone cushion incorrectly positioned and not properly flush with the breast shield		Ensure that the silicone cushion is completely flush with the breast shield
No vacuum can be built up		Ensure that the breast shield is completely flush with the breast
Pump suction too strong	Suction strength is set too high	Turn the suction strength controller in the "-" direction

Problem	Reason	Remedy
Pump makes unusual noises	Lip valve cannot process the amount of milk flowing	Reduce suction strength. If milk flow is high, a low suction strength is adequate to express efficiently
	The pump will make a rattling noise, if shaken, which comes from the automatic cut-out.	This rattling noise is normal and has no affect on the pump's function.

13. Technical Data

Medical product according to directive 93/42/EEC	Class I
NUK Teat NUK Breast Milk Container	Comply with EN 14350
Temperature range - Operation - Storage	+8 °C to +40 °C +8 °C to +40 °C
Power supply: - Mains operation via adapter Type KS07-045-0800 or KSD10-045-0800 or Y06FE-045-0800G or Y10DE-045-0800G - Battery operation	Input 100 – 240 V~/ 50/60 Hz, Output 4.5 VDC, 800 mA, hollow plug: polarity inner +, outer – 3 Type AAA batteries, 1.5 V, not rechargeable

14. Disposal

In the European Union, the symbol of a crossed-out wheeled bin means that the product must be disposed of at a separate waste collection point. This applies to this product and to all accessories marked with this symbol. Products labelled in this way must not be disposed of with normal household waste, but should be taken to a collection point for recycling electric and electronic equipment.

15. Warranty

Subject to careful treatment of your appliance and observance of these operating instructions, the manufacturer offers a 24 month warranty from the date you purchase the product.

Appliances sent to us postage paid will be repaired free of charge in the event of faulty materials or workmanship. Any further claims are in accordance with your statutory rights.

No compensation will be given for damage caused by improper use of the device.

Wear of the cable or plug is excluded from the warranty.

Within the first 24 months after the date of purchase we will rectify any recognised faults without any charge for incidental costs incurred.

Under all circumstances, the device must be returned accompanied by this completed warranty certificate or, failing this, the purchase receipt or other proof of the purchase date must be supplied.

If no proof of purchase date is provided, you will be invoiced for the costs incurred without further inquiry, and the item will be returned to you with a request of payment on delivery.

Please complete the warranty form immediately after purchase and keep this, or the till receipt, as proof of purchase in a safe place. Lost warranty certificates will not be replaced.

Please keep the address, Art. No. and LOT No. in case of complaints. The LOT No. can be found in the battery compartment and is marked "LOT xxxx".

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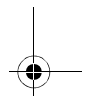
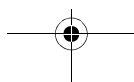
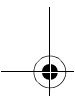
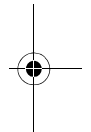
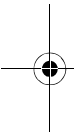
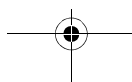
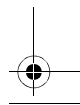
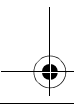
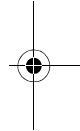
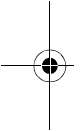
EMC-Guidance






Warnings:

- Medical electrical equipment needs special precautions regarding EMC. The following EMC guidelines must be observed during installing and put into service of the device.
- Portable and mobile RF communications equipment can affect medical electrical equipment.

Guidance and manufacturer's declaration - electromagnetic emissions

The electrical breast pump NUK Luna is intended for use in the electromagnetic environment specified below. The customer or the user of the electrical breast pump NUK Luna should assure that it is used in such an environment.

Emission test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 14	complies	The electrical breast pump NUK Luna is not suitable for interconnection with other equipment.
Harmonic emissions IEC 61000-3-2	Class A	The electrical breast pump Luna is suitable for use in all establishments, including domestic establishments and those directly connected to the public-low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations / flicker emissions IEC 61000-3-3	complies	

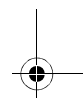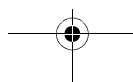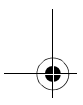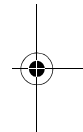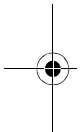Guidance and manufacturer's declaration - electromagnetic emissions			
The electrical breast pump NUK Luna is intended for use in the electromagnetic environment specified below. The customer or the user of the electrical breast pump NUK Luna should assure that the breast pump is used in such an environment.			
Immunity test	IEC 60601-1 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	+/- 6kV contact +/- 8 kV air	+/- 6kV contact +/- 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic materials, the relative humidity should be at least 30%.
Electrical fast transient/ burst IEC 61000-4-4	+/- 2 kV for power supply lines +/- 1 kV for input/output lines	+/- 2 kV for power supply lines +/- 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	+/- 1 kV differential mode +/- 2 kV common mode	+/- 1 kV differential mode +/- 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% for 5s <5% for 0,5 cycles 40% for 5 cycles 70% for 25 cycles	<5% for 5s <5% for 0,5 cycles 40% for 5 cycles 70% for 25 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of the electrical breast pump NUK Luna requires continued operation during power mains interruption, it is recommended that the electrical breast pump NUK Luna be powered from an uninterruptible power supply or battery.
Power frequency (50/60Hz) magnetic fields IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of the typical location in a typical commercial or hospital environment.

Guidance and manufacturer's declaration - electromagnetic emissions			
The electrical breast pump NUK Luna is intended for use in the electromagnetic environment specified bellow. The costumer or the user of the electrical breast pump NUK Luna should assure that it is used in such an environment.			
Immunity test	IEC 60601-1 test level	Compliance level	Electromagnetic environment - guidance
conducted RF IEC 61000-4-6	3 Veff 150 kHz to 80 MHz	3 V	Portable and mobile RF communications equipment should be used no closer to any part of the Public Access Defibrillator (COR 043), including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended Separation distance: $d = 1,17 * \text{SQRT}(P/W)$
Radiated IEC 61000-4-3	3 V/m 80 MHz to 2,5 GHz	3 V/m	$d = 4 * \text{SQRT}(P/W)$ for 80 MHz to 800 MHz $d = 7,67 * \text{SQRT}(P/W)$ for 800 MHz to 2,5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strength from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the symbol "nonionising radiation".
NOTE: At 80 MHz and 800 MHz the separation distance for the higher frequency range applies.			
NOTE : This guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
Field strength from fixed transmitters, such as base stations for radio (Cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy.			
To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the electrical breast pump NUK Luna is used exceeds the applicable RF compliance level above, additional measures may be necessary, such as reorientation or relocating the electrical breast pump NUK Luna.			

Recommended separation distances between portable and mobile RF communications equipment and the electrical breast pump NUK Luna






The electrical breast pump NUK Luna is intended for use in an electromagnetic environment in which the radiated RF disturbances are controlled. The customer or the user of the electrical breast pump NUK Luna can help prevent electromagnetic interference by maintaining a minimal distance between portable and mobile RF communications equipment (transmitters) and the electrical breast pump NUK Luna as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter P in W	Separation distance according to frequency of transmitter in m		
	80 MHz to 800 MHz	800 MHz to 2.5 GHz	150 KHz to 80 MHz
	$d=1,17 * \text{SQRT}(P/W)$	$d=2,33 * \text{SQRT}(P/W)$	$d=1,17 * \text{SQRT}(P/W)$
0,01	0,12	0,23	0,12
0,1	0,37	0,74	0,37
1	1,17	2,33	1,17
10	3,70	7,37	3,70
100	11,70	23,30	11,70

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE: At 80 MHz and 800 MHz the separation distance for the higher frequency range applies.

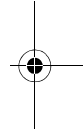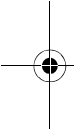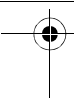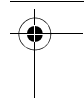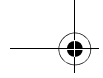NOTE : This guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

MAPA GmbH
Industriestraße 21-25
27404 Zeven
Germany
www.nuk.com

 Matr.-Nr. 10.520.192
REV 300/04-2013

NUK is a registered trademark of MAPA GmbH/Germany